ANAESTHILION®

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ANAESTHILION[®] Chloroethyl (ethyl chloride) spray

LEAFLET FOR THE USER

Read this leaflet carefully before you start using this medical device because it contains important information for you. If you have any questions ask your doctor or a specialist or your pharmacist.

ANAESTHILION[®] - chloroethyl (ethyl chloride) spray, Fine and Direct versions.

Composition: Active substance: chloroethyl (alternative names: chloroethane, ethyl chloride, C₂H₅Cl).

Content, Packing: Liquid chloroethyl – clear colourless liquid, spray bottle volume 100 ml or 175ml.

Application: Local anaesthetic. The spray is a vapocoolant to be applied to the skin to gain local anaesthesia (undercooling). The **Fine version** has a fine spray when applied to the skin and thus affects a wider area of the skin, i.e. also around the area to be numbed. The **Direct version** sprays the current more precisely, during application it sprays a narrow stream and thus hits exactly the part of the skin on which the current falls, without the wider area.

INDICATIONS / APPLICATION AREAS:

Anaesthilion is intended for local anaesthesia through undercooling (by freezing), for temporary skin anaesthesia e.g. before injection, in surgery, dermatology (in small surgical procedures such as abscesses, removal of warts and foreign particles), piercing, removal of ticks, muscular convulsions, sharp irritation of sinews (tennis elbow). Furthermore, it can also be applied to sports injuries of soft tissues.

APPLICATION:

Apply by spraying onto the affected area from a distance of about 10-20 cm until a white snow film forms. Do not extend the spray application time unnecessarily as this may cause frostbite! The effect lasts short time. **DO NOT INHALE** vapours! Protecting the adjacent skin areas with a thin petroleum jelly layer or a cloth during the spraying procedure is recommended. The thawing process may be painful, and freezing may reduce local resistance to infection and delay healing. When applying in a close proximity to the patient's face is necessary to cover his/her eyes, nose and mouth to prevent him/her from inhaling the product.

CONTRAINDICATIONS:

When not to use Anaesthilion?

Anaesthilion should not be used on individuals who are allergic to chloroethyl or who have a history of hypersensitivity to it. Local anaesthetics cannot be applied to the following areas: open wounds, broken skin and eczema, mucous membranes such as inside the mouth or nose, on the mouth or lips, the ears, nose or eyes, the anus or genitals. Do not use before or during dialysis! For external use only! Do not spray into the eyes! Avoid inhaling the product! Inhalation of chloroethyl must be avoided as it may produce narcotic and general anaesthetic effects. Anaesthilion is not intended to suppress internal pain such as back, etc.

PREGNANCY/BREASTFEEDING/PAEDIATRIC USE:

No controlled safety studies on pregnant women or children are available, and so, caution should be exercised when using this product on such patients. Chloroethyl may generally be used on children as well as adults. However, when using the product on a child, the doctor should be very observant, not to overspray when the skin blanches. Children's skin may be more sensitive to the low temperatures than adult skin. Using chloroethyl on children under 6 years age is not recommended.

INTERACTIONS:

Since the product is applied externally and evaporates fast, no interactions with other medicinal products are known. The product may react negatively by contact with some tools or instruments, especially if they are plastic, so it cannot be applied to anything other than the skin surface of the human body. Applying Anaesthilion to any instrument or device and then applying it to your skin is not effective, so do not use it! Anaesthilion does not have analgesic effects such as a number of products such as mesocain, xylocain, etc.

INCORRECT APPLICATION AND OVERDOSE:

Excessive chloroethyl application may induce frostbites. Tissue congelation can increase the risk of infection. Symptoms of intoxication on aspiration of chloroethyl may include the following: blurred vision, stupor, intoxication, unconsciousness, and even paralysis of the respiratory and circulation systems. Countermeasures, in addition to standard procedures, are aimed at helping pulmonary excretion of the toxic substance by forced exhalation and by preventive administration of xylocaine (ventricular fibrillation). In view of the possible development of ventricular fibrillation, do not administer sympathomimetics! Do not use alcohol, milk or castor

oil! If frostbite develops on the application (skin may become red), this may be due to improper application or hypersensitive skin.

SIDE EFFECTS:

Allergic changes may appear on the skin. No side effects were observed where chloraethyl had been used in compliance with instructions in this package leaflet. Frostbite may develop if chloraethyl was sprayed too long (excessively). Some patients with sensitive skin exhibit red skin, the colour, however, disappears in a few hours or days. If you get any side effects, talk to your doctor or pharmacist. This includes any possible side effects not listed in this leaflet. Freezing the skin may occasionally temporarily change the color of the skin's pigmentation (the site in question will have a different shade of color than the surrounding parts), especially if it was brown after sunbathing. Therefore, we do not recommend the use of Anaesthilion on tanned skin.

Adverse reactions: Skin hypersensitivity may occur rarely. Other adverse reactions are not known.

What precautions must be taken when driving, using machines or working in unstable positions?

No adverse effects on drivers or machine operators are expected if the product is used as described in this package leaflet. However, if you have inhaled chloraethyl, your reflex capacity may be reduced insomuch that you are unable to drive or use machines. See the P phrases and H phrases on the label for additional or more detailed instructions.

PRECAUTIONARY AND HAZARD STATEMENTS:

H222 Extremely flammable aerosol. H229 Pressurized container: May burst if heated. P102 Keep out of reach of children. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. – No smoking. P251 Do not pierce or burn, even after use. P211 Do not spray on an open flame or other ignition source. P270 Do not smoke when using this product. P260 Do not breathe vapours. P262 Do not get in eyes. P243 Take precautionary measures against static discharge.

Storage and shelf life:

P410+412 Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F. P403+P271 Store and use in a well ventilated place. Store at room temperature 10-25°C.

Do not use this product after the expiry date which is stated on the bottom of the bottle. Keep in a dark and cold place at room temperature (10-25°C) protected from direct sunlight and other thermal sources. Spontaneous evaporation is an intrinsic property of ethyl chloride gas and minor gas leakage during the product storage is no problem.

Note: In case of adverse effects/events contact directly the manufacturer at info@btcpharma.cz Information regarding product withdrawal from the market/use will be published at the manufacturer's website, www.btcpharma.cz.

Date of the latest revision of the instructions for use: 03/2023.

Sole agent/distributed by: ACHILLEION PHARMA

Manufacturer: BTC Pharma, s.r.o., Bořivojova 35/878,
13000 Praha 3, Czech Republic, info@btcpharma.cz
Tel. +420603511030.

Danger

Warning